IDEXX SmartQC* Control

Choose your language

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Selezione seu idioma

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Zvolte jazyk

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IDEXX SmartQC* Control

Intended use and summary

IDEXX SmartQC* Control is designed for the automated quality control analysis and statistical processing of the ProCyte One* Hematology Analyzer. This product is intended for veterinary use only.

Formulation

IDEXX SmartQC Control consists of synthetic polymer beads with optical properties that simulate blood cells. The control consists of beads of two different sizes and a dye to simulate hemoglobin.

Storage and stability

- Smart QC Control vials should be stored at room temperature (15°C–35°C [59°F–95°F]) to ensure product stability.
- When properly stored, unused IDEXX SmartQC Control vials are stable until the expiration date on vial.
- IDEXX SmartQC Control is stable for 180 days after it is installed in the analyzer.

Precautions and warnings

- H317
- P231 / P272 / P280 / P302+P350 / P333+P313 / P363
- May cause an allergic skin reaction.
- Avoid opening the cap so that you do not breathe any mist/vapors.
- Wear protective gloves when using IDEXX SmartQC Control.
- If contact with skin occurs, wash affected area with soap and water.
- Wash contaminated clothing before reuse.

Disposal

Dispose of IDEXX SmartQC vials in accordance with your local/regional/national/international regulations.

Instructions for use

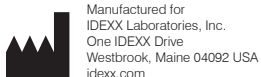
1. When prompted, tap **Replace IDEXX SmartQC**.
2. Open the sample drawer and remove the existing vial.
3. Shake a new IDEXX SmartQC vial vigorously for 10 seconds to ensure it is well mixed and then load it into the sample drawer.
4. Close the sample drawer. The analyzer automatically enters the QC lot information from the barcode on the vial label into the IDEXX VetLab* Station.

Indication of product deterioration

- Using the material without sufficient mixing can result in bead agglutination, potentially impacting the accuracy of the results.
- When the vial is opened, there is a potential for fluid evaporation from the vial, which can lead to incorrect count statistics.

Limitations

- IDEXX SmartQC Control is for use on the ProCyte One Hematology Analyzer.
- IDEXX SmartQC Control is to be used as supplied; dilution invalidates intended use.

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IDEXX Laboratories, Inc.
IDEXX Europe B.V.